

PURPOSE:

This document contains important installation information for installing the VolumePro 1000 External Power Supply Cables.

WARNING

PLEASE READ ENTIRE DOCUMENT TO PREVENT DAMAGE!

EQUIPMENT:

None

GENERAL NOTES:

When installing the VolumePro 1000 External Power Supply cables:

Plan your installation carefully.

The cables are rated for interior use only.

Check to assure that the cables are installed without twists or kinks.

Check to assure good mechanical and electrical contact at all connectors and plugs.

It is suggested that the cables not be allowed to rest against sharp or metallic objects as it could wear through the jacket compromising its integrity and possibly causing an electrical connection.

Please respect the stiffness of the black mains cable to the External Power Supply and the black low voltage cable from the External Power Supply when handling.

WARNING

DO NOT make sharp twists, bends or kinks to black mains cable to the External Power Supply and the black low voltage cable from the External Power Supply especially near the connectors to prevent possible damage to the cables interior conductors.

Doing so may void your warranty.

BENDING:

DO NOT BEND CABLES UNDER THE MINIMUM BEND RADIUS OF 4X DIAMETER OF CABLE.

AT THE CONNECTOR ENDS, WITHIN 12 INCHES (OR ~30 CM) OF THE CONNECTORS DO NOT BEND OR TWIST THE CABLE CONNECTORS OR PLUGS OVER 30 DEGRESS IN ANY DIRECTION.



Examples of proper VolumePro 1000 External Power Supply Cables installation



Examples of improper VolumePro 1000 External Power Supply Cables installation